


CODIAGNOSTIX® **INSTALLATION** **AND ACTIVATION**

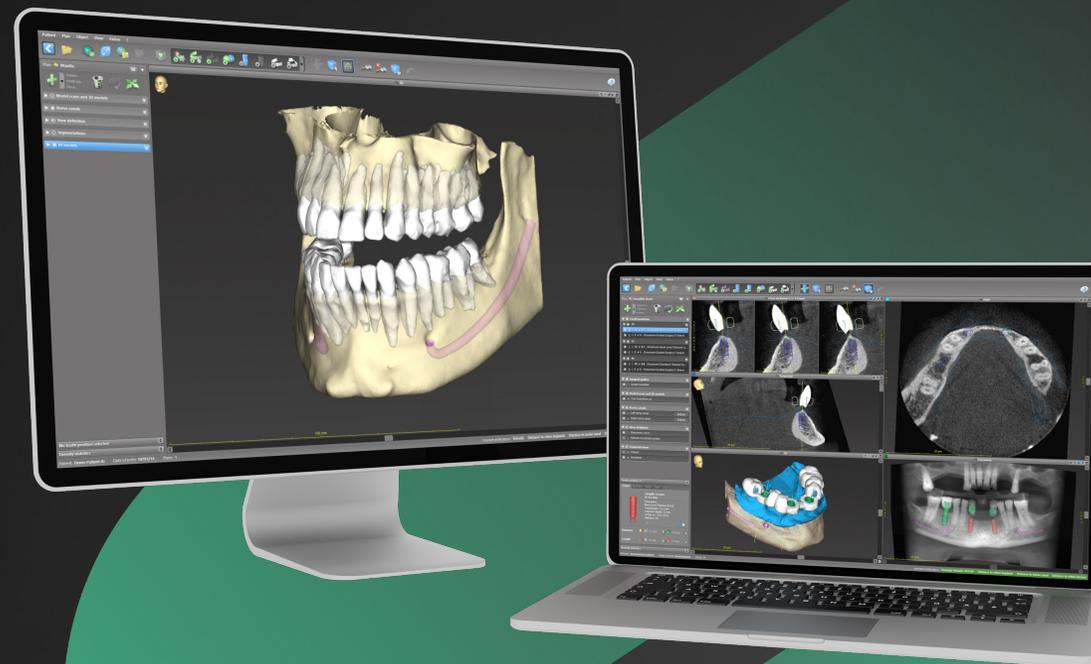


CODIAGNOSTIX® INSTALLATION AND ACTIVATION

1 DOWNLOAD


At the end of your subscription order, you received an **email** with a **link** to download coDiagnostiX®. If you cannot find this email, please check your spam folder.

Download the coDiagnostiX® installer and locate coDiagnostiX_Installer_xxxx.exe on your computer.

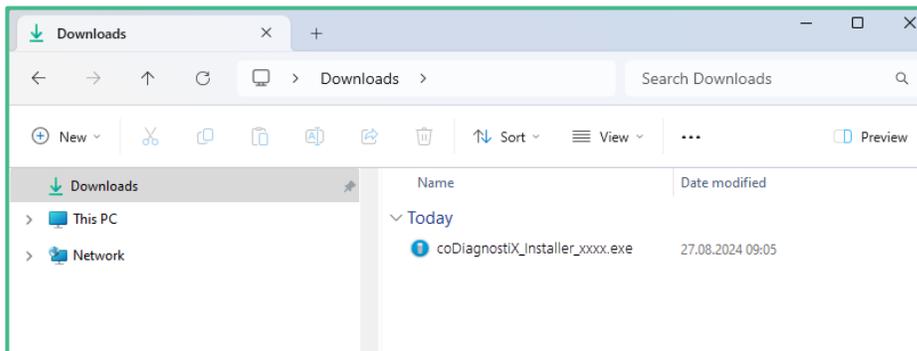

2 INSTALLATION


Double-click coDiagnostiX_Installer_xxxx.exe to start the coDiagnostiX® installation process. Select your language and click **OK**.

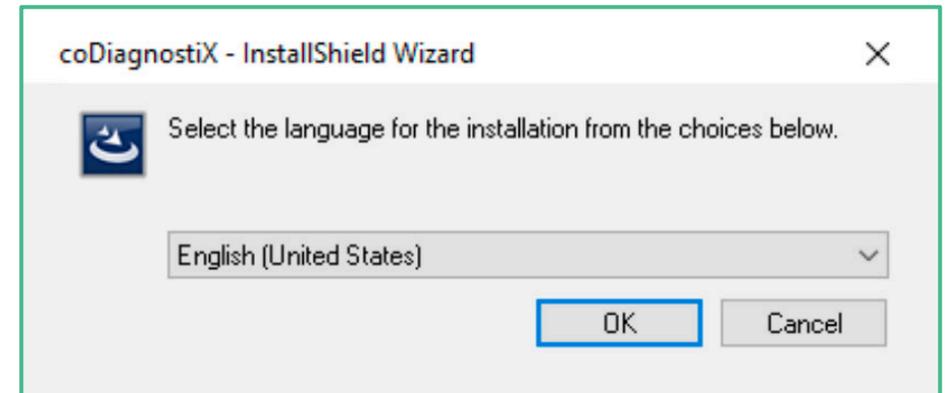

CODIAGNOSTIX® INSTALLATION AND ACTIVATION

3 INSTALLATION

Follow the prompts to install coDiagnostiX®.

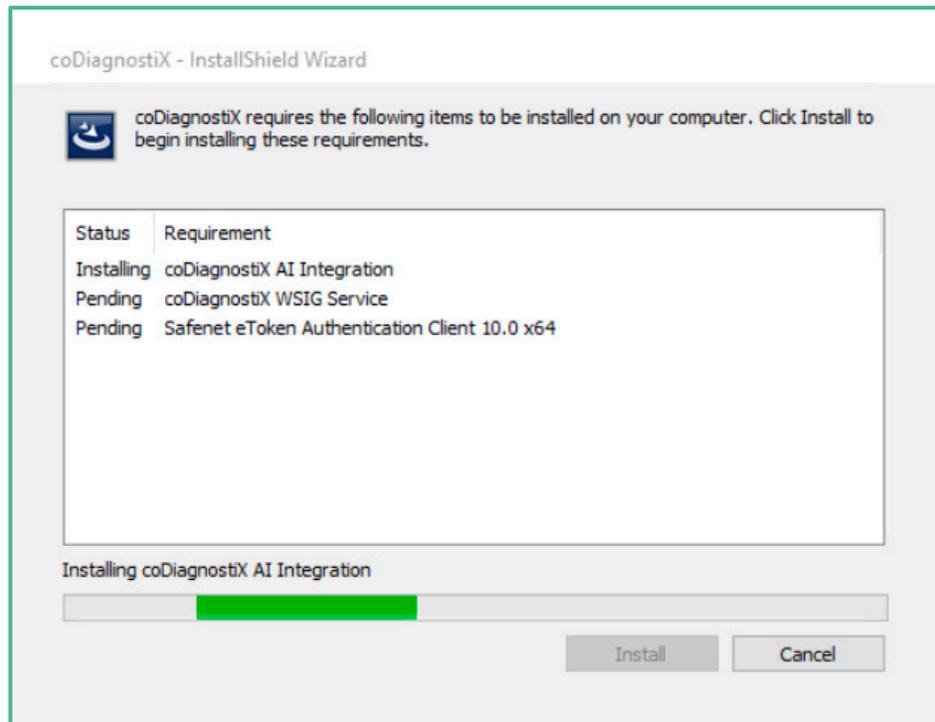

4 INSTALLATION

Once complete, select **Finished** and locate coDiagnostiX® on your desktop.



5 ACTIVATION


Start coDiagnostiX® by double-clicking the coDiagnostiX® icon on your desktop.

CODIAGNOSTIX® INSTALLATION AND ACTIVATION

6 ACTIVATION


Once the software runs, open the **Straumann® AXS License Management**:

A. VIA THE LICENSE CHECK POP-UP WINDOW

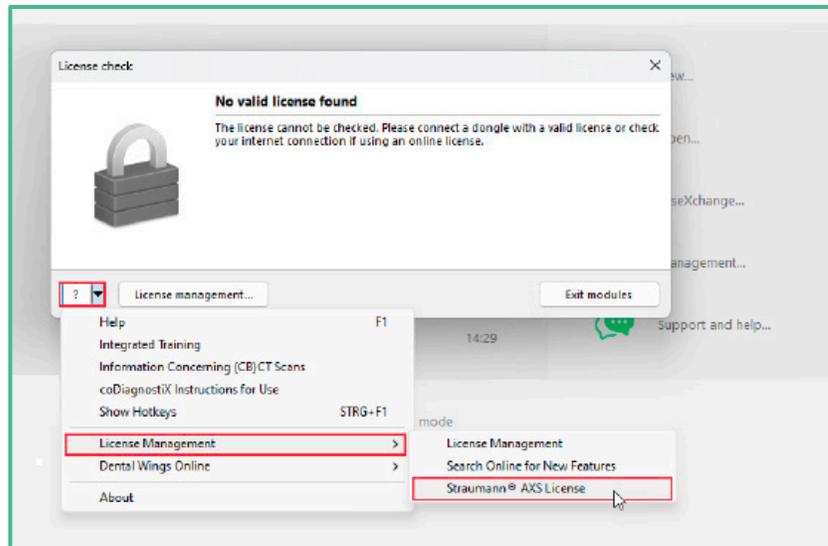

By clicking on ? and selecting **License Management** and then **Straumann® AXS License**.

B. VIA THE CODIAGNOSTIX® MAIN WINDOW

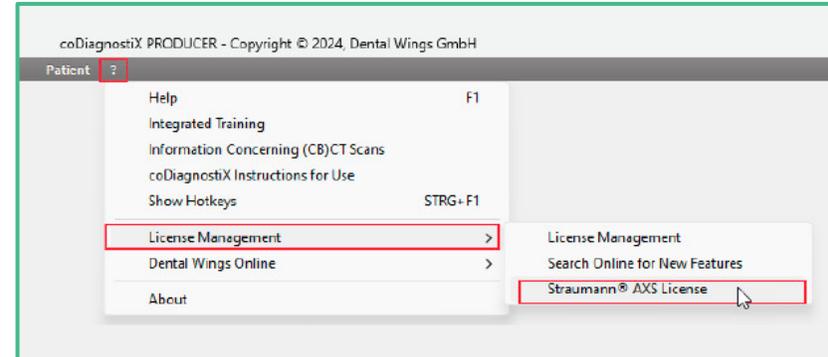


By clicking on ? in the top menu bar and selecting **License Management** and then **Straumann® AXS License**.

CODIAGNOSTIX® INSTALLATION AND ACTIVATION


7 ACTIVATION

Read the **information** regarding **AXS license management** and click **Next**.

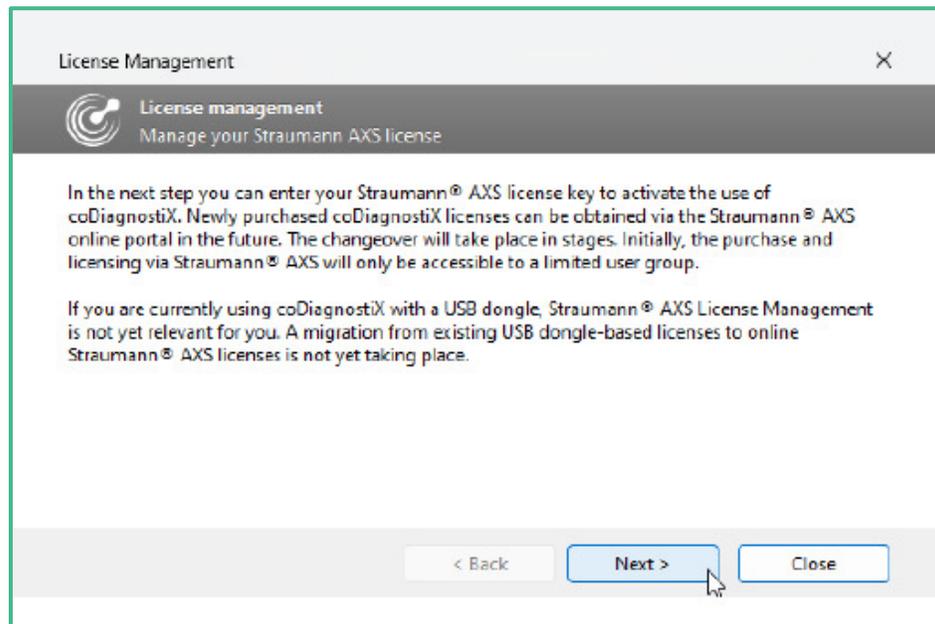

8 ACTIVATION


Access your **AXS account** and **copy** your **personal key**, which you can find on the **Subscription** page.

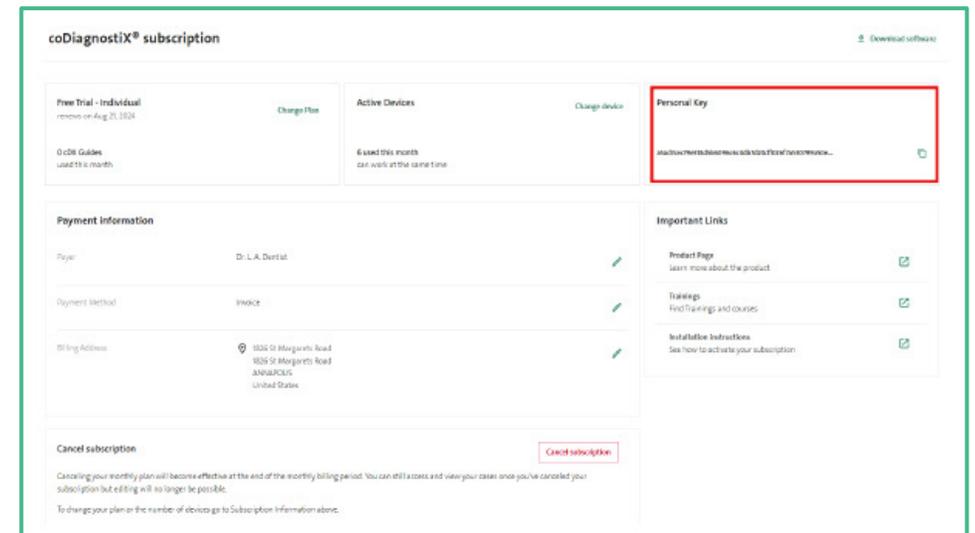

CODIAGNOSTIX® INSTALLATION AND ACTIVATION

9 ACTIVATION

Paste this key in the License Management dialog box of coDiagnostiX®.

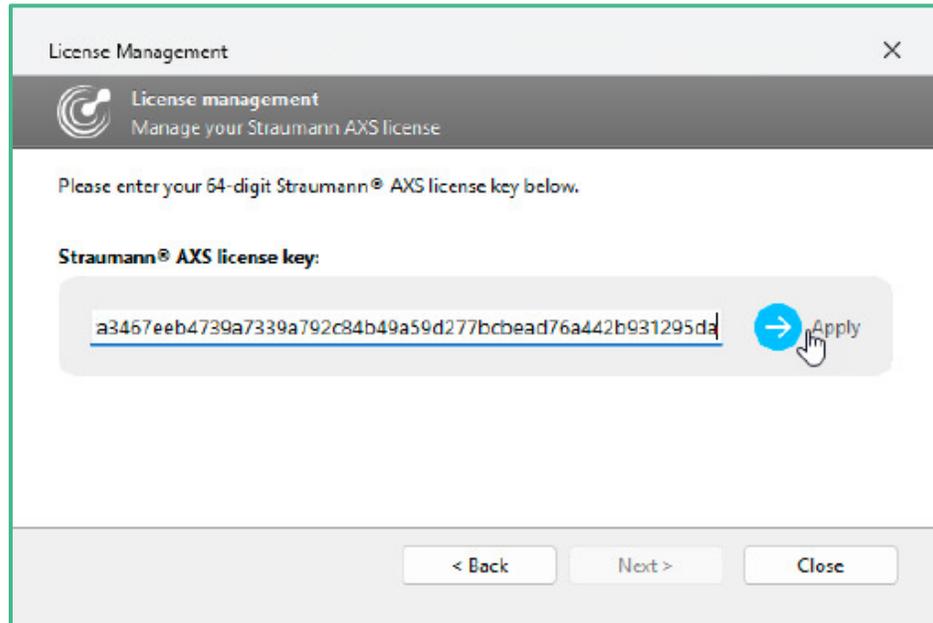

10 ACTIVATION

Once your coDiagnostiX® license is **successfully activated**, you can **start working** with the software.

