

# coDiagnostiX® INSTALLATION PROCESS

Thank you for choosing coDiagnostiX®. Please follow this guide to complete the installation process. If you require assistance at any time, please contact us at [codiagnostix.com/support](https://codiagnostix.com/support)



1

coDiagnostiX® components are located in the case.



2

The case contains:

- Dongle
- USB drive
- IFU information




3

Remove the USB drive and insert it into your computer's USB port



4

In your computer's folders find the cDX Install drive. Inside you will find the application called setup.exe

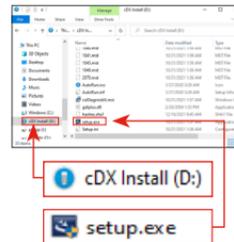

5

Double click on setup.exe to start the coDiagnostiX® install process. Select your language and hit OK.

